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ELANEE Brustmassage-Gerät

- D** ELANEE Brustmassage-Gerät – Gebrauchsanweisung.....2
- GB** ELANEE Breast Massager device – Instructions for use.....5
- F** Appareil de massage des seins ELANEE – Mode d'emploi.....8
- I** Massaggiatore per il seno ELANEE – Istruzioni per l'uso.....11

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Instructions for Use ELANEE Breast Massager device

Table of Contents

1. Scope.....	15
2. Product contents.....	15
3. Safety instructions	16
4. Description of the device.....	18
5. Intended and appropriate use.....	19
6. Range of usage.....	19
7. Operation.....	19
8. Usage.....	20
9. Safekeeping, service life, storage and transport.....	22
10. Cleaning and disposal instructions.....	22
11. Troubleshooting.....	23
12. Warranty and Service.....	24
13. Material composition and technical specifications.....	24
14. Explanation of symbols used.....	25

Dear Customer,

Thank you for purchasing our ELANEE Breast Massager device! These Instructions for Use should help you to get the best use out of your product. Please read the instruction manual carefully - particularly the “safety instructions” - before using this product for the first time. You can download the Instructions for Use from our website, at www.elanee.de.

We will be glad to answer any questions and to receive any suggestions you might have.

Your *ELANEE Team*

 Please follow the instructions for use and any notes on the packaging and keep both the instructions and the packaging!

1. Scope

These instructions for use apply to the following product:

REF 223-00 ELANEE Breast Massager device

2. Product contents

- 1 ELANEE Breast Massager device
- 1 USB charging cable
- 1 Instructions for Use
- 1 ELANEE storage bag

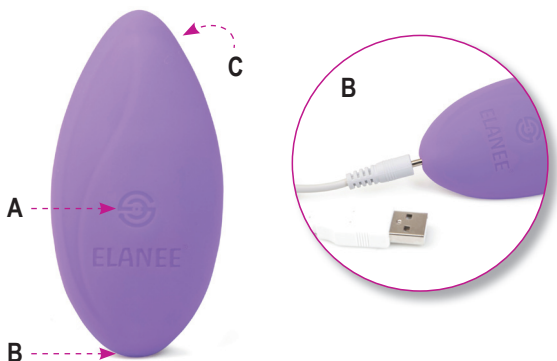
3. Safety instructions

-  The device is a vibrating massager device for female breasts during pregnancy and breastfeeding.
-  This device can be used by children aged 8 and over as well as by people with reduced physical, sensory or mental capabilities or with a lack of experience and knowledge so long as they are supervised or instructed about the safe use of the device and understand the dangers that can result from using it.
-  Children must not play with the device.
-  Cleaning and user maintenance must not be carried out by children without supervision.
-  The device must only be operated with safety extra-low voltage (SELV) from a low power circuit according to EN 60335-1 labelling on the device.
-  The device has an in-built battery which contains dangerous substances for the environment. Before disposing of the device, the battery must be removed from it. To do this, disconnect the USB charging cable. Open the case to remove the battery. The battery can then be safely removed.

Alternatively, ensure an environmentally friendly disposal of the device or of the battery according to battery regulations.
-  Please check your ELANEE Breast Massager device before each use. Only use a device that is in perfect condition (with no damage or sharp edges).
-  Please charge your device for 8 hours before using it for the first time.

-  Please only use the USB charging cable included in the packaging to recharge your device.
-  Charge the device at room temperature.
-  Disconnect the USB charging cable when the device is not being charged.
-  If the device or USB charging cable is damaged, do not connect it to a power source. Contact your electronics retailer or customer service department.
-  Incorrect use or the disregarding the instruction manual could lead to a risk of fire.
-  The use of non-original manufacturer approved accessories can damage the product.
-  Prolonged usage is not recommended. If the device becomes warm, switch it off and allow it to cool down before using it again. Heat-sensitive people are asked to take care.
-  In the case of known hypersensitivities and allergies to one of the mentioned substances, the device should not be used or should only be used after having consulted a doctor.
-  In the case of injuries, swellings or redness, usage of the device should be stopped. If in doubt, ask your doctor or midwife.
-  Pacemaker wearers must consult their doctor before using the device.
-  The device should not be left near unsupervised babies or children and it should be protected from heat as well as kept dry.

4. Description of the device

- A. On and off button (press and hold for longer than 5 seconds) and button for programme selection (press and release)
- B. Opening for USB charging cable
- C. Area of contact on the breast

Programme 1	continuous light
Programme 2	continuous medium
Programme 3	continuous strong
Programme 4	pulsing light
Programme 5	pulsing medium
Programme 6	pulsing strong

5. Intended and appropriate use

The ELANEE Breast Massager device is a vibrating massaging device suitable for the female breast during pregnancy and breastfeeding. The shape of the device has been designed to suit the shape of the breast.

The Breast Massager device provides a pleasant massage and promotes blood supply in the breast tissue. The stimulated blood circulation can help to better take care of the breast tissue. The milk flow is stimulated and blockages can be relieved.

The Breast Massager device has six different programme levels which differ by strength and pulse type. The user should choose the programme which gives them the physical comfort they need.

The ELANEE Breast Massager device is not suitable for any purposes other than the intended uses listed.

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The ELANEE Breast Massager device is suitable for women during pregnancy and whilst breastfeeding. The device is particularly suitable for women who are breastfeeding for the stimulation of milk flow and/or to relieve blockages in the breast.

The device is exclusively intended for you to use yourself and for non-professional use.

7. Operation

The ELANEE Breast Massager device is suitable for a gentle massage of the breast. The device is placed on the breast using the hand and moved in small circles or from all directions from the edge of the breast to the nipple. Thanks to the six programmes with various vibration strengths, you can decide which vibration strength is best for you at that time.

Regular massages using the ELANEE Breast Massager device can help to relax the breast tissue and lead to improved breastmilk flow.

8. Usage

1. Fully charge the Breast Massager device (for 8 hours) before the first use by inserting the DC connector into the appropriate opening on the device and connecting the USB charging cable to the power source (adapter for a power socket or USB port on the computer):

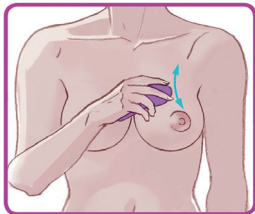
The LED display flashes when the device is charging.

The LED display is continuously illuminated when the device is fully charged.

2. Switch the device on by keeping the on/off button pressed for longer than 5 seconds.
3. Select a programme and change programmes as necessary by briefly pressing the on/off button.
4. Hold the device in your hand and place it directly on your breast or over your clothes. You can wear normal clothes for the massage (preferably a top made of thin material or just a bra).
5. Perform stroking or circular movements from the edge of the breast to the nipple.
6. You can just massage the breast for comfort or you can express at the same time, depending on what feels better for you.
7. Repeat steps 5 to 8 on the other breast and perform the massage until the blockages are relieved.
8. Switch the device off by keeping the on/off button pressed for longer than 5 seconds.


Tip: If you need your hands free, you can also place the device directly in your bra. The length and intensity of the massage can be adjusted according to your personal comfort.

Three different recommendations for the breast massage:


1. Place the device on the top of the breast, stroke it downwards towards the nipple:

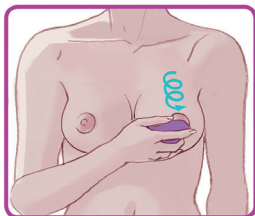Start on the upper outer edge of the breast and with a soft and steady pressure slide the massage device in the direction of the nipple - use inwards stroke movements around the breast.

2. Place the device on the top of the breast, make circular movements towards the nipple:

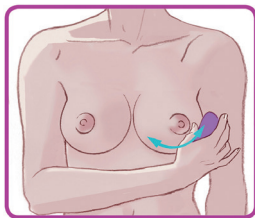Start on the upper outer edge of the breast and with a soft and steady pressure create circles using the massage device - use inwards circular movements around the breast.

3. Place the device on the lower part of the breast, stroking movements from left to right:

Place the massage device on the lower outer edge of the breast and with a soft and steady pressure slide it along the breast from left to right and from right to left.

9. Safekeeping, service life, storage and transport

Please store the ELANEE Breast Massager device in a dry place, protected from sunlight – ideally, in its packaging.

Keep the device out of the reach of children.

The Breast Massager device is intended for reuse and under normal use has a service life of at least 4 years (minimum battery life: 600 to 800 charge cycles).

10. Cleaning and disposal instructions

The device should only be cleaned with a damp cloth.

If the USB charging cable port becomes dirty, clean this also only with a damp cloth.

The device must not be disposed of with normal household waste. Disposal should be made in accordance with the WEEE (Waste Electrical and Electronic Equipment) European Directive using appropriate collection points in your country.

The device should only be taken to a disposal point in a fully discharged condition otherwise precautions must be taken against short circuits if batteries are not fully discharged.

If you have questions regarding waste disposal, please contact your local authorities responsible for waste disposal.

Please separate the individual packaging components and follow the relevant recycling system.

Please be aware of local disposal guidelines.

11. Troubleshooting

Problem	Cause	Solution
The device is not (1) switching on or (2) switching off.	(1) The device is not sufficiently charged.	Please fully charge your device (see Point 8 on Usage).
	(2) The on/off button was not pressed down for long enough.	Please press the on/off button for longer than 5 seconds.
	The device is faulty.	Please send your device back to us to the address given in Point 12.
The device makes noises during usage.	The device's massage function is based on vibrations which are produced through an in-built battery. For this reason noises are normal.	No action is needed.
The programme stops during usage.	The maximum running time was reached.	Please fully charge your device (see Point 8 on Usage).
	The device is faulty.	Please send your device back to us using the address given in Point 12.

12. Warranty and Service

We provide a 2 year warranty from the date of purchase for material and manufacturing defects of the product. The warranty, however, does not apply:

- in the case of damage due to incorrect use,
- to wear parts,
- in the case of customer negligence, and
- if the device has been opened or repaired by an unauthorised shop.

Customers' statutory warranty claims remains unaffected by the warranty. To submit a warranty claim within the warranty period, proof of purchase must be provided by the customer. A warranty claim can be made within a period of 2 years from the date of purchase against GRÜNSPECHT Naturprodukte GmbH, Karlskron (Germany).

In case of complaints, please contact us as follows:

Phone: +49 (0)8450-6803300, E-Mail: post@gruenspecht.de, www.gruenspecht.de

Faulty, fully charged products should be sent to the following address:

GRÜNSPECHT Naturprodukte GmbH, Münchener Str. 21,
85123 Karlskron, Germany
(Subject to errors and alterations)

13. Material composition and technical specifications

The ELANEE Breast Massager device is made from:

- Polysiloxane (silicone without phthalates)
- Silicon dioxide
- Platinum complex

14. Explanation of symbols used

	Manufacturer
	Batch number
	Item number
	Follow Instructions for Use
	Store in a dry place
	Protect from sunlight
	Attention!
	CE conformity marking
	Specialist disposal of electrical appliance
	Specialist disposal of the battery
	Device with protection class III
	Direct current
	Polarity of the charging socket

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